GUIDED | **KIT** **DAS** SURGICAL

3shape 
Implant Studio

INSTRUCCIONES DE USO
DYNAMIC ABUTMENT SOLUTIONS LIBRARY

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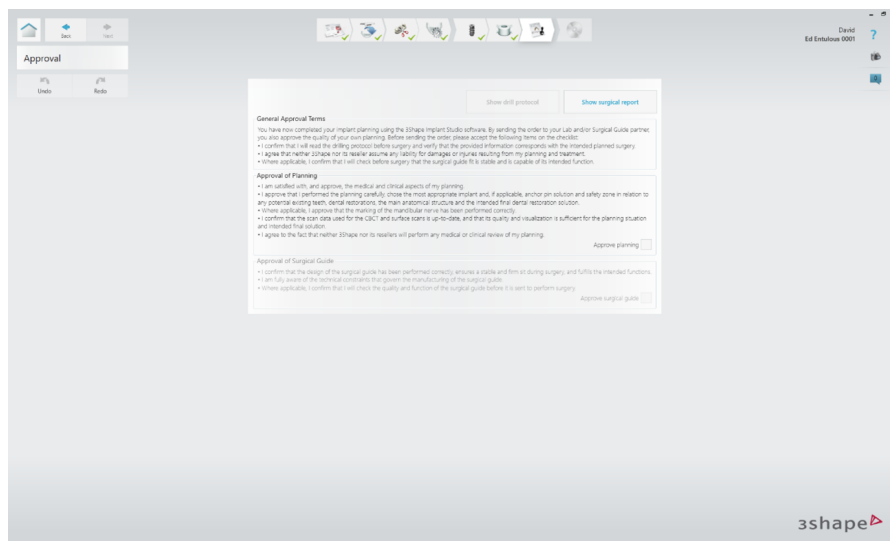
Show drill protocol Show surgical report

General Approval Terms

You have now completed your implant planning using the 3Shape Implant Studio software. By sending the order to your Lab and/or Surgical Guide partner you also approve the quality of your own planning. Before sending the order please accept the following items on the checklist:

- I confirm that I will read the drilling protocol before surgery and verify that the provided information corresponds with the intended planned surgery.
- I agree that neither 3Shape nor its reseller assume any liability for damages or injuries resulting from my planning and treatment.
- Where applicable, I confirm that I will check before surgery that the surgical guide fit is stable and is capable of its intended function.

Approval of Planning

- I am satisfied with, and approve, the medical and clinical aspects of my planning.
- I approve that I performed the planning carefully, chose the most appropriate implant and, if applicable, anchor pin solution and safety zone in relation to any potential existing teeth, dental restorations, the main anatomical structure and the intended final dental restoration solution.
- Where applicable, I approve that the marking of the mandibular nerve has been performed correctly.
- I confirm that the scan data used for the CBCT and surface scans is up-to-date, and that its quality and visualization is sufficient for the planning situation and intended final solution.
- I agree to the fact that neither 3Shape nor its resellers will perform any medical or clinical review of my planning.

Approve planning

Approval of Surgical Guide

- I confirm that the design of the surgical guide has been performed correctly, ensures a stable and firm fit during surgery, and fulfills the intended functions.
- I am fully aware of the technical constraints that govern the manufacturing of the surgical guide.
- Where applicable, I confirm that I will check the quality and function of the surgical guide before it is sent to perform surgery.

Approve surgical guide

After approving the implant planning you will not be able to modify it. Do you want to continue?

Yes No

3Shape Implant Studio

Surgical Report

Order Details **APPROVED**

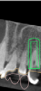
Patient Name: 64 Estelinos
 Client Order Reference: 25176345_20221216_1382_49
 Creation Date: 16/12/2022 14:09:05
 Approved by: David David.david@3shape.com
(2020-Head-soft-soft-3SWP-Workshop2)



Implant Information		12	13
Implant position (POS)			
Manufacturer	Generic Implants	Generic Implants	Generic Implants
Type	Generic 3Shape_Genetic_2.0-D-3.9-mm-8.0-3.7	Generic 3Shape_Genetic_2.0-D-3.9-mm-8.0-3.7	Generic 3Shape_Genetic_2.0-D-3.9-mm-8.0-3.7
Order number	N/A	N/A	N/A
Length, mm	8	8	8
Diameter (D), mm	3.7	3.7	4.2
Color	White	White	White

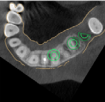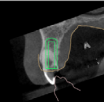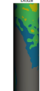
Implant Information		12	13
Implant position (POS)			
Manufacturer	Generic Implants	Generic Implants	Generic Implants
Type	Generic 3Shape_Genetic_2.0-D-3.9-mm-8.0-3.7	Generic 3Shape_Genetic_2.0-D-3.9-mm-8.0-3.7	Generic 3Shape_Genetic_2.0-D-3.9-mm-8.0-3.7
Order number	N/A	N/A	N/A
Length, mm	8	8	8
Diameter (D), mm	4.2	4.2	4.2

Implant Information	
Implant position (POS)	12
Manufacturer	Generic Implants
Type	Generic 3Shape_Genetic_2.0-D-3.9-mm-8.0-3.7
Order number	N/A
Length, mm	8
Diameter (D), mm	3.7
Color	White
Guide pin - axial distance	3.0
Guide pin - radial distance	1.5

Show drill protocol Show surgical report

General Approval Terms

You have now completed your implant planning using the 3Shape Implant Studio software. By sending the order to your Lab and/or Surgical Guide partner you also approve the quality of your own planning. Before sending the order please accept the following items on the checklist:

- I confirm that I will read the drilling protocol before surgery and verify that the provided information corresponds with the intended planned surgery.
- I agree that neither 3Shape nor its reseller assume any liability for damages or injuries resulting from my planning and treatment.
- Where applicable, I confirm that I will check before surgery that the surgical guide fit is stable and is capable of its intended function.

Approval of Planning

- I am satisfied with, and approve, the medical and clinical aspects of my planning.
- I approve that I performed the planning carefully, chose the most appropriate implant and, if applicable, anchor pin solution and safety zone in relation to any potential existing teeth, dental restorations, the main anatomical structure and the intended final dental restoration solution.
- Where applicable, I approve that the marking of the mandibular nerve has been performed correctly.
- I confirm that the scan data used for the CBCT and surface scans is up-to-date, and that its quality and visualization is sufficient for the planning situation and intended final solution.
- I agree to the fact that neither 3Shape nor its resellers will perform any medical or clinical review of my planning.

Approve planning

Approval of Surgical Guide

- I confirm that the design of the surgical guide has been performed correctly, ensures a stable and firm fit during surgery, and fulfills the intended functions.
- I am fully aware of the technical constraints that govern the manufacturing of the surgical guide.
- Where applicable, I confirm that I will check the quality and function of the surgical guide before it is sent to perform surgery.

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Contact information: **3Shape Implant Studio**

Drilling Protocol

Order Details


Patient Name: Ed Entulous

Client Order Reference: 25176345_20221216_1102_49

Creation Date: 16/12/2022 14:03:00

Created by: David - david.lopez@dynamicalabutment.com (208c4bed-68d8-480f-b597-fe952dbca3c)

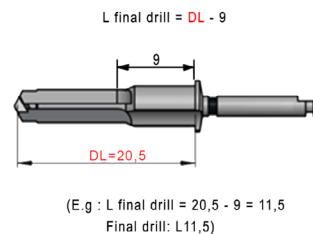Implant information	
Implant position (FDI)	26
Manufacturer	Generic Implants
Type	Generic IPGS_Generic_SHORT_4.0-4.9_mm 10.0H.2
Order number	N/A
Length, mm	10
Diameter (Ø), mm	4.2
Color	White
Sleeve information	
Name	Dynamic Abutment Solutions Sleeve D 4.85
Type	Fully guided
Order number	71.340.485.02-2
Offset, mm	10,5
Color	Purple
Drill information	
Minimum drill length	20,5

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